

**Skin has  
many stories  
to TELL.  
You just  
need to be  
able to  
READ them.**



## HEINE Dermatoscopes



	DELTA 20 T	iC 1	NC 2	mini 3000 LED	mini 3000
Polarisation	Polarised + non-polarised (switchable)	Polarised + non-polarised (switchable)	Polarised	Non-polarised (immersion)	Non-polarised (immersion)
Illumination	LED <sup>HQ</sup>	LED <sup>HQ</sup>	LED <sup>HQ</sup>	LED <sup>HQ</sup>	XHL
Magnification	10 to 16x	Up to 40x digital	6x/ 10x Up to 30x digital	10x	10x
Battery/charging options	<ul style="list-style-type: none"> <li>Battery handle</li> <li>USB rechargeable handle</li> <li>Rechargeable handle for table charger</li> </ul>	USB cable	USB cable	<ul style="list-style-type: none"> <li>AA batteries</li> <li>Rechargeable handle for table charger</li> </ul>	<ul style="list-style-type: none"> <li>AA batteries</li> <li>Rechargeable handle for table charger</li> </ul>
Contact plate	<ul style="list-style-type: none"> <li>Contact plate with scale</li> <li>Contact plate without scale</li> </ul>	<ul style="list-style-type: none"> <li>Contact plate with scale</li> </ul>	<ul style="list-style-type: none"> <li>Contact plate with scale</li> </ul>	<ul style="list-style-type: none"> <li>Contact plate with scale</li> <li>Contact plate without scale</li> </ul>	<ul style="list-style-type: none"> <li>Contact plate with scale</li> <li>Contact plate without scale</li> </ul>
Photographs possible?	Yes, with optional SLR photo adaptor	Yes, with iPhone/iPod touch	Yes, with adaptor for iPhone/iPod touch	–	–
Optional accessories	Small contact plate for difficult-to-access lesions	–	–	Small contact plate for difficult-to-access lesions	Small contact plate for difficult-to-access lesions
Catalogue page	093 / 097	094 / 097	095 / 098	096 / 098	096 / 098

The Apple iPhone or iPod touch is not included in the scope of delivery. Apple, the Apple logo, iPhone and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

## HEINE DELTA® 20 T Dermatoscope

### You can't treat what you can't see

**LED<sup>HQ</sup>**  
LED NOW IN HEINE QUALITY.



[01]

- ∴ **Crystal clear, high-resolution images** thanks to the high-quality achromatic optical system. The focusing system delivers 10 to 16x magnification for a detailed view of even the smallest structures.
- ∴ **Toggle function for switching between polarised and non-polarised illumination mode.** With the press of a button. No exchange of contact plates necessary. The additional diagnostic function allows detection of 'blink signs' when viewing crystalline structures and milia.
- ∴ **Glare-free and reflection-free view thanks to polarisation, even without immersion fluid.** For fast and flexible work.
- ∴ **With just one hand:** individual focusing, range of correction approximately -6 to +6 dpt.
- ∴ HEINE has set a new standard which maintains that only the best is good enough, from the selection of materials to processing, from light intensity and thermal management to a colour rendering index as high as possible.  
**That's what we call LED in HEINE Quality – or LED<sup>HQ</sup>.**
- ∴ **The DELTA20 T is also capable of digital documentation:** using the HEINE digital camera lens system, the dermatoscope can be connected to the SLR cameras of major brands without any issues. Specially developed photo adaptor lenses combine the camera and DELTA20 T to create an unbeatable pair.



HEINE DELTA 20 T film



- ∴ **Special small contact plate for the examination of difficult-to-access lesions, for instance between the fingers or toes.** See page 097 for order information.

★ **TOPSELLER**



[02]

★

DELTA 20 T Dermatoscope Kits	LED
<b>Kit 1:</b> DELTA 20 T Dermatoscope with contact plate with scale, BETA 4 USB rechargeable handle with USB cord and plug-in power supply, without hard case [02]	<b>K-010.28.388</b>
<b>Kit 2:</b> DELTA 20 T Dermatoscope with contact plate with scale and BETA 4 NT rechargeable handle, without hard case [01]	<b>K-010.24.396</b>
<b>Kit 3:</b> 2x DELTA 20 T Dermatoscope with contact plate with scale, 2x BETA 4 NT rechargeable handle and NT4 table charger, without hard case [03]	<b>K-012.24.422</b>

Brings structure to your desk: the innovative charging system charges two handles at the same time!



[03]



HEINE® iC 1 Dermatoscope

Dermatoscopy goes digital. And mobile.



Free  
HEINE iC 1 App



∴ **Sharp, sharper, sharpest.** The HEINE iC 1 is a device designed for capturing clinical and dermatoscopic images for documenting and supporting differential diagnosis of melanomas and other skin lesions. And in our unrivalled quality. Sharp and shadow-free.  
Thanks, on the one hand, to our precise, aspherical optical system and, on the other, the 12-megapixel image quality when using an iPhone®\* 6s or a later model. Simply download iC 1 App, insert your iPhone and experience digital dermatoscopy like never before.

∴ **Big, bigger, biggest.** The high-quality aspherical optical system with up to 40x true magnification offers everything you need in a practice.  
∴ HEINE has set a new standard which maintains that only the best is good enough, from the selection of materials to processing, from light intensity and thermal management to a colour rendering index as high as possible.

**That's what we call LED in HEINE Quality – or LED<sup>HQ</sup>.**

∴ **At the press of a button – the toggle function.** With two switching buttons, the option to switch back and forth between polarisation and immersion images is always within easy reach, regardless of the working position.

∴ **Many see a smartphone. We see an exceptional camera and first-class display.**

We've opted for the iPhone for reasons of quality, user orientation and costs. From our perspective it offers the best camera, a display with the highest resolution and the most progressive touchscreen technology.

\* Apple, the Apple logo, iPhone and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

★ TOPSELLER




<b>iC1 Set/5 for iPhone 5s and SE</b>	<b>LED</b>
iC 1 with contact plate with scale, mounting case smartphone iC 1/5 for Apple iPhone 5s and SE, USB cord with medical approved plug-in power supply	<b>K-271.28.305</b>



<b>iC1 Set/6 for iPhone 6/6s</b>	<b>LED</b>
See above, but with mounting case smartphone iC 1/6 for Apple iPhone 6/6s	<b>K-270.28.305</b>

<b>iC1 Set/7 for iPhone 7 and 8</b>	<b>LED</b>
See above, but with mounting case smartphone iC 1/7 for Apple iPhone 7 and 8	<b>K-273.28.305</b>

The Apple iPhone is not included in the scope of delivery.

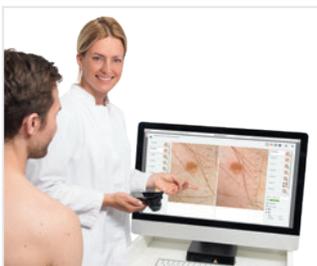

∴ **A WHOLE NEW DIMENSION OF SKIN SCREENING: HEINE CUBE SYSTEM.**

Image and data management for up to ten mobile, digital HEINE iC 1 dermatoscopes. Just one highly intelligent investment can help integrate patient data, manage images, monitor progression and increase patient loyalty – and makes work independent, convenient and intuitive. More information can be found on our website: [www.heine.com/en/digital](http://www.heine.com/en/digital)



## HEINE® NC 2 Dermatoscope

## The all-in-one dermatoscope

LED<sup>HQ</sup>  
LED NOW IN HEINE QUALITY.

- ∴ **Precise images.** The contact plate [01] achieves a clear, reflection-free and highly detailed image in 10× magnification, even without immersion fluid.
- ∴ **Non-contact.** Without the magnetically-held contact plate [02], the NC2 is used for non-contact examination in 6× magnification for hygienic and efficient skin examination with a large field of view.
- ∴ **Digital.** The optional iPhone®/iPod®\*\* accessory and the free NC2 app we've developed allow you to use the NC2 like a digital camera. With up to 30× digital magnification and up to 12-megapixel resolution.\*\* Including an extensive range of documentation options: integrated patient management, email function, classification of image information, and patient details on a bodymap. Even in non-contact mode.
- ∴ **Polarised illumination** for immediate examination without the use of contact fluids.
- ∴ HEINE has set a new standard which maintains that only the best is good enough, from the selection of materials to processing, from light intensity and thermal management to a colour rendering index as high as possible.  
**That's what we call LED in HEINE Quality – or LED<sup>HQ</sup>.**
- ∴ **An especially large overview field** thanks to HEINE precision optics (lens with 27 mm diameter) for crystal clear, high-resolution images.
- ∴ **Ocular adjustable** from 0 to +3 dpt for correction and refocussing.
- ∴ **Micro-USB connection** for charging the integrated Li-ion battery. The rechargeable battery can be replaced by authorised specialist dealers.

\* The Apple iPhone or iPod touch is not included in the scope of delivery. Apple, the Apple logo, iPhone and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

\*\* with iPhone from Version 6s

Free  
HEINE NC2 App

## NC2 Dermatoscope Sets

## LED



NC2 Dermatoscope (incl. USB handle), **contact plate with scale**, USB cord with E4-USB medical approved plug-in power supply, carry bag [01]

Please order mounting case smartphone separately, see accessories page 098.

D-841.78.120

See above, but **without contact plate**

D-842.78.120

## ★ TOPSELLER



[01]

- ∴ **HEINE NC2 Dermatoscopes are also suitable for use with the HEINE Cube System.**



## HEINE mini3000® LED Dermatoscope

### Even our smaller models are great



- ∴ **Modern pocket dermatoscope with handy dimensions:** small on the outside, but huge on the inside.
- ∴ HEINE has set a new standard which maintains that only the best is good enough, from the selection of materials to processing, from light intensity and thermal management to a colour rendering index as high as possible.  
**That's what we call LED in HEINE Quality – or LED<sup>HQ</sup>.**
- ∴ **Twice as bright** as a conventional XHL Xenon Halogen dermatoscope with maintenance-free LED<sup>HQ</sup> illumination – no longer a need to replace the light source.
- ∴ **Exclusive Battery Performance Indicator:** When the instrument is switched on, the maximum light intensity is shown, then the light intensity is regulated down to the level that corresponds to the current charge status of the battery so you know when your batteries need changing.
- ∴ **Fade-Out Feature:** brightness reduces slowly even with low residual capacity, for even more efficient use of dry or rechargeable batteries.
- ∴ **10x magnification** with focusing optics.
- ∴ **Instrument head made of metal.**
- ∴ **20,000 ON/OFF switch cycles guaranteed.**
- ∴ **Distortion-free image** with high resolution edge to edge thanks to high-quality, colour-neutral achromatic optical system.
- ∴ **Special small contact plate** for the examination of difficult-to-access lesions, for instance between the fingers or toes.
- ∴ **Handle in black or blue.**

★ TOPSELLER



mini3000 Dermatoscope Sets	2.5V LED
★ mini3000 LED Dermatoscope with contact plate <b>with scale</b> , 10 ml dermatoscopy oil, mini3000 Battery handle with batteries, zipper case	<b>D-888.78.021</b>
see above, but with contact plate <b>without scale</b>	<b>D-887.78.021</b>

mini3000 LED Dermatoscope	2.5V LED
on mini3000 battery handle with batteries, contact plate <b>with scale</b> and 10 ml dermatoscopy oil	<b>D-008.78.109</b>
see above, but with contact plate <b>without scale</b>	<b>D-008.78.107</b>

## HEINE mini3000® XHL Dermatoscope

The mini3000 with XHL illumination can do it all just as well as its little big brother with LED<sup>HQ</sup> (see above). Except with XHL illumination. Authentic colour rendering and even illumination thanks to concentrated, bright Xenon Halogen light.



mini 3000 Dermatoscope	2.5V XHL
on mini3000 battery handle with batteries, contact plate <b>with scale</b> , and 10 ml dermatoscopy oil	<b>D-001.78.109</b>
see above, but with contact plate <b>without scale</b>	<b>D-001.78.107</b>

Accessories for DELTA 20 T Dermatoscope



[01]

[02]



DELTA 20 T Dermatoscope Head	LED
with contact plate Ø 23mm <b>with scale</b> and dermatoscopy compendium, without handle	K-008.34.221
with contact plate Ø 23mm <b>without scale</b> and dermatoscopy compendium, without handle	K-008.34.222

Contact plates	
23mm dia. with scale [01]	K-000.34.216
23mm dia. without scale [01]	K-000.34.217
★ Small contact plate, 8 mm dia. [02] for the examination of difficult-to-access lesions	K-000.34.205

Dermatoscopy oil	
(pack of 6 x 10 ml). Reduces reflexes on the outer layers of skin.	K-000.34.005

SLR Photo adaptor	
for Dermatoscope/digital SLR camera, with optics. Wide Field of View, crisp image	
for Canon	K-000.34.185
for Nikon	K-000.34.186

Adaptor cord	
(1.3m) for instrument/handle	X-000.99.231

Photo Accessory Set	
includes SLR Photo adaptor, adaptor cord, distance ring and BETA Belt Clip	
for Canon	K-000.34.190
for Nikon	K-000.34.191

BETA Belt Clip	
for BETA handle	X-000.99.008

Accessories for iC 1 Dermatoscope



[01]



[02]

Accessories	
Contact plate with scale [01]	K-000.34.201
Mounting case smartphone iC 1/7 for Apple iPhone 7/8 [02]	K-000.34.253
Mounting case smartphone iC 1/6 for Apple iPhone 6/6s	K-000.34.250
Mounting case smartphone iC 1/5 for Apple iPhone 5s/SE	K-000.34.251
iPod mounting case iC 1/T6 for Apple iPod touch 6th generation	K-000.34.252
USB cord with E4-USB medical approved plug-in power supply	X-000.99.303



Accessories for NC 2 Dermatoscope



Accessories	
Contact plate with scale [01]	D-000.78.114
Mounting case smartphone NC 2/7 for Apple iPhone 7/8 [02]	D-000.78.127
Mounting case smartphone NC 2/6 for Apple iPhone 6/6s	D-000.78.126
Mounting case smartphone NC 2/5 for Apple iPhone 5s/SE	D-000.78.125
iPod mounting case NC 2/T6 for Apple iPod touch 6. Generation	D-000.78.124
USB cord with E4-USB medical approved plug-in power supply	X-000.99.303

Accessories for mini3000 LED / XHL Dermatoscope



mini3000 LED Dermatoscope Head	
with contact plate <b>with scale</b> , without handle	D-008.78.108
with contact plate <b>without scale</b> , without handle	D-008.78.106

mini 3000 XHL Dermatoscope Head	
with contact plate <b>with scale</b> , without handle	D-001.78.108
with contact plate <b>without scale</b> , without handle	D-001.78.106

Contact plates	
with scale [01]	D-000.78.117
without scale [01]	D-000.78.116
Small contact plate, 8mm dia. [02] for the examination of difficult-to-access lesions	K-000.34.008

Dermatoscopy oil	
Pack of 6x 10ml. Reduces reflexes on the outer layers of skin.	K-000.34.005

XHL Xenon Halogen spare bulb	
for mini3000 Dermatoscope	X-001.88.109

Zipper case	
for mini3000 Dermatoscope	D-887.00.000